



If there is no sign displayed, then Print this Notice at the Kingston Parish Website and search for Document KPC Notice Defibrillator Records and Checklist

To Print this Notice, go to the Kingston Parish Website and search for Document KPC Notice KPC Notice Defibrillator Records and Checklist.docx



Defibrillator Checklist

1. The Defibrillator will Self-Test regularly. Check for any visible and or audible warnings on the devise indicating that there is a Fault.
 2. Check that the devise is in Standby Mode.
 3. Check that the Devise is Clean and not Damaged.
 4. Check the Battery Indicator.
 5. Ensure that the Battery and Pads have not Life Expired.

Cabinet Checklist

1. Check for Damage.
2. Check that the Defibrillator is easily accessible.



Defibrillator and Cabinet Maintenance Record

Defibrillator

Location : Kingston Village Hall

Frequency of Check : Every Two Months

Make / Model : iPad CU-SP1

Serial Number : G1Q29B428

Manufactured Date : 13/09/2013

Battery Serial Number : GV371253 D

Battery Manufactured Date : 09/2023

Battery Expiry Date : 09/2027

Pads Lot Number : Y060321-07

Pads Manufactured Date : 06/2021

Pads Expiry Date : 06/2024



Cabinet

Location : Kingston Village Hall

Frequency of Check : Every Two Months

Make / Model : DefibSafe

Serial Number : AGA460

Manufactured Date : March 2019



East of England Ambulance Service

NHS Trust

DEFIBRILLATOR REGISTRATION FORM

Site Address (to Inc county & post code): Kingston Village Hall, The Green Kingston, Cambridge, Cambridgeshire CB23 2NN	
Location of defibrillator (and cabinet if appropriate): <i>(This should include as much information as possible to identify the position of the defibrillator such as: To the right hand side of the main entrance to - or - located behind reception etc.)</i>	
Location of defibrillator (and cabinet if appropriate): On outside of side wall of village hall, to the right of the main entrance. Location code on cabinet: AGA460	
Make and model of defibrillator:	WelMedical iPad CU-SP1
Serial number of defibrillator: <i>(To include ambulance service asset number if appropriate):</i>	G1Q29B428 13/09/2018
Owner of defibrillator: <i>(Please state if owned by Ambulance Service or community/village/charity etc.).</i>	Kingston Parish Council
Expiry date of pads:	06/2021
Date when defibrillator was purchased:	11/03/2019
Battery serial number:	?
Battery manufacture date:	?
Make and colour of cabinet:	Defibsafe 2 – Green and yellow
Is your cabinet heated: Yes/No	Yes
Is your CPAD site accessible 24/7: Yes/No	Yes
Key code for the cabinet: <i>(if installing an unlocked cabinet please state UNLOCKED):</i>	Unlocked
Custodian contact details:	
Name: Peter Stokes Phone Number: 01223262207, 07740053247 Email Address: kingstonpc.cambridge@gmail.com	Custodian Address: The Old Post Office, Rectory Lane, Kingston, Cambridge CB23 2NL
If you require any help or support please contact the office via the email address below Once complete please email the registration form to: defibs@eastamb.nhs.uk Or post to: Defibrillator Registrations, The Ambulance Station, Owen Road, Diss, Norfolk IP22 4ER	

The Data Protection Act 2018

As an NHS Trust we will not disclose your personal details to third parties without your explicit permission, unless there are exceptional circumstances such as; when it is in the public interest to investigate a serious crime, or to prevent serious risks to the public or NHS staff; or to protect children or vulnerable adults or where the law requires information to be shared through a formal court order. If the Trust does have to pass on any of your personal information to a third party it will follow the guidance and application of the Data Protection Act 2018. The Legal Basis for processing your personal data is with your explicit consent. You have the right to withdraw your consent at any point by informing the Community Collaboration Team at the above address and or the Data Protection Officer informationgovernance@eastamb.nhs.uk

To Print this Notice, go to the Kingston Parish Website and search for Document KPC Notice KPC
Notice Defibrillator Records and Checklist.docx